

MEDICAL VACUUM FILTERS




Walker Filtration design and manufacture in the UK a comprehensive range of medical vacuum filters for centralised hospital vacuum plant installations as specified in the UK standard HTM2022. The Walker range of medical vacuum filters are designed to protect these installations from liquid, solid and bacterial contamination. Liquids are collected in a transparent drain flask which can be easily removed for sterilisation.

Models A27-A308 incorporate the unique Walker designed 'push-on' filter element. This reduces maintenance time and allows the filter to be located in the most confined spaces. The MV grade filter elements incorporate a pre-filter and low pressure loss filter to remove solid and bacterial contamination. They use high efficiency borosilicate glass microfibre media to remove all


dirt particles. All elements include stainless steel metalwork and are fitted with an external pre filter layer of 80p.p.i., open cell reticulated polyester foam. These filters are a proven success and now include such features as differential pressure indicators which are a specific requirement of the HTM2022 medical gas pipeline specification.

WALKER
FILTRATION

FILTER MODEL	PIPE SIZE	FREE AIR CAPACITY AT ATMOSPHERIC PRESSURE			RAREFIED AIR CAPACITY AT 500 mm Hg VACUUM			ELEMENT MODEL	NUMBER OF ELEMENTS	DIMENSIONS mm				WEIGHT		DIMENSIONS inch				FILTER MODEL
		L/min	SCFM	Nm ³ /h	L/min	SCFM	Nm ³ /h			A	B	C	D	Kg	lb	A	B	C	D	
A27 MV	1/4	30	1	1.8	90	3	4.2	E511 MV	1	88	32	315	100	1.0	2.2	3 1/2	2	13	4	A27 MV
A39 MV	3/8	100	3.5	6	300	10.5	18	E511 MV	1	88	32	315	100	1.3	2.9	3 1/2	2	13	4	A39 MV
A56 MV	1/2	200	7	12	600	21	36	E521 MV	1	88	32	315	100	1.5	3.3	3 1/2	2	15	4	A56 MV
A70 MV	3/4	250	9	15	750	27	45	E521 MV	1	88	32	365	100	1.5	3.3	3 1/2	2	15	4	A70 MV
A105MV	1	600	21	36	1800	63	108	E811 MV	1	125	39	365	100	2.7	6.0	5	2	15	4	A105MV
A126 MV	1 1/4	1200	42	72	3600	126	216	E821 MV	1	125	39	565	150	3.5	7.7	5	2	21	6	A126 MV
A153 MV	1 1/2	2000	70	120	6000	210	360	E831 MV	1	135	50	545	200	4.4	9.7	5 1/2	2	22	8	A153 MV
A205 MV	2	3000	106	180	9000	318	540	E851 MV	1	135	50	745	200	5.0	11.0	5 1/2	2	30	8	A205 MV
A306 MV	3	4000	141	240	12000	423	720	E1261 MV	1	200	68	925	200	15.5	34.2	7 7/8	3	37	8	A306 MV
A308 MV	3	5000	177	300	15000	531	900	E1281 MV	1	230	65	1050	300	19	41.9	9	3	42	12	A308 MV
A391 MV	DN 80	5500	194	330	16500	582	990	E139 MV	1	450	300	940	700	58	128	17 3/4	12	37	28	A391 MV
A483 MV	DN 100	6765	238	406	20295	714	1218	E88 MV	3	520	300	960	700	74	163	20 1/2	12	38	28	A483 MV
A484 MV	DN 100	8200	289	492	24600	867	1476	E88 MV	4	520	300	960	700	74	163	20 1/2	12	38	28	A484 MV
A686 MV	DN 150	11000	388	660	33000	1164	1980	E88 MV	6	680	400	1000	700	165	364	26 3/4	16	40	28	A686 MV

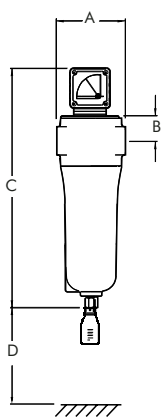

A27 MV to A308 MV

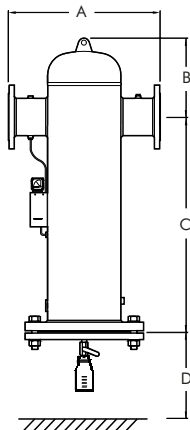

A391 MV to A686 MV

SPECIFICATION	Grade MV	
Penetration to BS 3928	< 0.0001%	
Maximum temperature	100°C	212°F
Pressure loss - clean	35 mbar	0.5 psi
Pressure loss - change element	100 mbar	1.5 psi
Maximum working pressure	7 barg (see note 10)	100 psig (see note 10)
Maximum working vacuum	1013 mbar	15 psi
Element end cap colour	black	

NOTES

1. Threaded medical vacuum filters are manufactured from aluminium.
2. Threaded connections are BSP parallel to ISO7/1 or NPT to ANSI B2.1 if supplied within North America.
3. Filter element end caps are colour coded black. Direction of air flow, outside to in through filter element.
4. Mounting brackets are available for models A27 MV to A308 MV.
5. All aluminum medical vacuum bodies are electrophoretically painted and then coated with blue polyester powder paint finish to eliminate corrosion.
6. Flanged medical vacuum filters are fabricated from carbon steel.
7. Flanged medical vacuum filters are designed and manufactured in accordance with BS EN286 and carry the CE mark.
8. Flanged connections complete with mating flange to BS 4504 PN16.
9. All flanged medical vacuum bodies are coated inside and out with blue polyester powder paint finish to eliminate corrosion.
10. Manual drain valves are fitted to all models. Sterilisable glass drain flasks are supplied as standard, 100 ml for models A27 to A70 and 250 ml for A105 to A686.
11. Differential pressure gauge, 55DPG250W is fitted to models A27 to A308, 65DPG250W is fitted to models A391 to A686.
12. Flow rates shown above are at free air conditions when operating at 500 mm hg.
13. WARNING - Pressure for test purposes MUST only be applied with drain valve CLOSED and drain flask REMOVED.



Walker Filtration Ltd
 Spire Road, Glover East,
 Washington, Tyne & Wear,
 NE37 3ES England.
 Tel: +44 (0)191 417 7816
 Fax: +44 (0)191 415 3748
 sales@walkerfiltration.co.uk
 www.walkerfiltration.com

Walker Filtration PTY Ltd
 3A Kia Court,
 Preston, Victoria 3072,
 Australia.
 Tel: +61 (0) 3 9480 0750
 Fax: +61 (0) 3 9480 1044
 sales@walkerfiltration.com.au
 www.walkerfiltration.com.au

Walker Filtration Inc
 2220 West 50th Street,
 Erie, PA 16506,
 USA.
 Tel: +1 814 836 2900
 Fax: +1 814 836 7921
 info@walkerfiltration.com
 www.walkerfiltration.com

Distributed by :